
## User Manual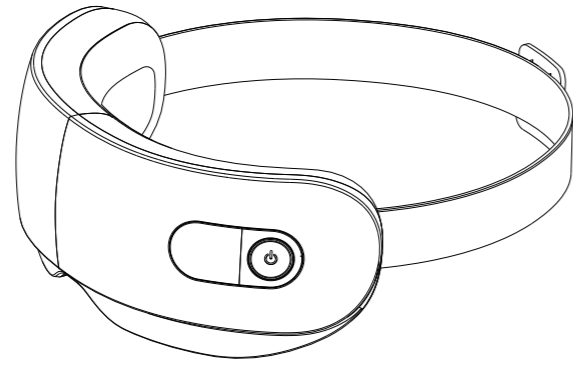

**Eye Massager**  
Model: MGE-1822

### Contents

English	01-06
Français	07-12

Shenzhen Bestvap Technology Co., Ltd.  
Web: www.naipocare.com  
E-mail: bestvap@naipocare.com  
Add: 503 ManTao Building 720 Minzhi  
Avenue Minzhi Street Longhua District,  
Shenzhen, Guangdong 518000

### Introduction

Thank you for purchasing the NAIPO Eye Massager. Please read this user manual carefully and keep it in a safe place for future reference. If you need any assistance, please contact our support team with your product model number and Amazon order number.

### Included

- 1 \* Eye Massager
- 1 \* USB Charging Cable
- 1 \* Storage Bag
- 1 \* Users Manual

### Product Specifications

Model	MGE-1822
Name	Eye Massager
Rated Power	5V--- (Recommend Using USB Adaptor > 1A)
Mode	SLEEP/MED/HARD
Battery Capacity	1500mAh

### \*Please Read The Users Manual Carefully Before Use.

### Important Safety Precautions


- Do not wear accessories around your eyes when using this product.
- For those with symptoms of eye disease, patients with major eye diseases such as eye surgery, glaucoma, cataract, and retinal detachment, please DO NOT use this product.
- For those who have medical electronic devices implants, such as cardiac pacemakers, or patients with malignant tumors, heart disease, or severe hypertension, please consult your physician before using this product.
- Children and mentally/physically disabled can only use the product with special supervision of a responsible adult.

- 01 -


- If you experience bodily pain and discomfort during use, please stop using it immediately and consult your doctor.
- This product is not a toy and can not be given to children for fun.
- This product is for single-person use. Do not operate simultaneously on more than 1 person each time.
- This product is only for home use. It is not a piece of medical equipment and is not intended for self-diagnosis, treatment, or other commercial purposes.
- Please charge the product using only accompanied adaptor and please be careful while you charge.
- Please cut off the power when the product exhibit malfunctioning. Do not disassemble or fix the product on your own.
- Do not use the product if you have a fever or body temperature higher than 38°C.
- Please do not disassemble the product on your own to prevent product malfunctioning.
- Do not use a power source other than the rated power to prevent fire, physical injuries, or product damages.
- Do not damage, destroy, over-bend, pull, twist, or tie the power adaptor.
- Do not expose the product in the environment of high-temperature or hot water.
- The product should not be discarded with regular home disposal. Please follow the local guideline for disposing of the product.
- For questions or inquiries, please email our customer support at support@naipocare.com.
- The direction of connecting the product or the USB power adaptor depends on the product model. Please do not force to connect to the product in case of damaged or shorted connected that may lead to fire or product malfunctioning.

- 02 -

### Product Diagram

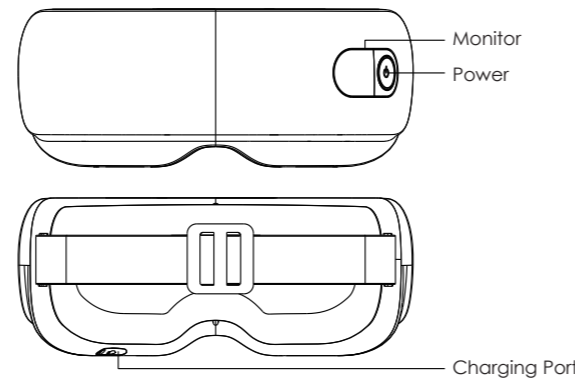### Monitor Display
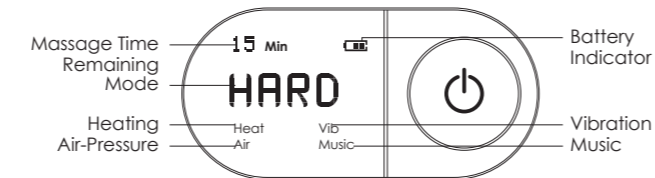
### Operation Instructions

- Using**
- Power On: In OFF condition, long press the Power button, the product will be activated after a beep sound.
  - Mode Switch: The default mode is MED after the product is powered on. Short press the Power button to switch. In looping sequence, between MED, HARD, SLEEP.
- SLEEP Mode (Air-pressure), 15min/round.

- 03 -

- MED Mode (Air-pressure+Music), 10min/round.
  - HARD Mode (Air-pressure+Heating+Vibration+Music), 15min/round.
  - Music: The MED and HARD modes are built-in with default music.
- Continuous pressing 3 times the Power button to turn the music off.**
- Power Off: Long press the Power button and the product would switch off after a beep sound.
  - Battery: When the product battery is insufficient, the battery symbol will start to flash.
  - Charging: During charging, the battery symbol would indicate that the product is charging. When the indicator remains lit, meaning that the charging is complete.
- \*Note: Charging Safety System: The product can not be activated during charge. If the product is plugged into the power during the working condition, it will automatically shut off due to the safety system).

### Charging

- Plug the USB adaptor with the charging port (Please take special note about the direction for connecting the Micro USB adaptor).
- After being connected to an appropriate power source, the battery indicator located on the top right corner of the monitor will start to indicate the charging progress.
- Please disconnect the product with the power source as soon as possible after the charging is complete. It takes roughly about 1.6 hours for a full charge.
- Please charge the product with a stable power source, otherwise, the battery capacity would be too low, thus extending the charging time.
- We recommend using USB adaptor with rated power larger than 1A.

### Cleaning & Maintenance

- Please turn off the power before cleaning the product.
- Please wipe the product with a soft cloth.
- If the product is very dirty, you can wipe it with cotton fabrics or cotton balls dampened with alcohol.
- Do not use water or corrosive liquids to clean the product.

- 04 -

### Storage

Please place the product inside of the storage bag with its original package and store it in a safe place.  
Do not put heavy weights on top of the product.  
Please take moisture, rodent and pollution prevention measures before storing the product.  
For product in more than 3 months of storage, please recharge before use.  
Do not store the product in places with direct sunlight, or corrosive gas.

### Troubleshooting

Problem	Reason	Solution
Can not turn on.	Power button wasn't pressed. Insufficient battery.	Press the Power button. Please recharge.
Can't charge.	Adaptor is not connected properly.	Check and reconnect the power adaptor.
Indicator light is weak or flashing. Air-pressure strength is weak.	Insufficient battery.	Please recharge.
Stopped working during operation.	Massage timer is off. Battery is out.	Press the Power button to reactivate the product. Recharge the battery.

- 05 -

### Warranty & Customer Support

If you need help regarding product issues, after-sales service, or other relevant problems, please contact us via the email on the cover.

\*Note: Products purchased directly from NAIPO benefit from our after-sales service. If the purchases are made from other selling teams, we recommend you contact them for warranty issues.

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